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Audibase 5.2 Unregistered

File Setup Edit Database Connect to Help

Register Details Audiograms Questionnaire

Patient No	Surname	Forenames	Last Test	Time of Test	Previous Test	Date of Birth	Categorisation	Position/Rank	Dept/Unit	Company	Location	S
80171700	Luis	Casas	27/07/2018	12:09:58 pm	21/05/2018	16/03/1982	HSE 2006: 1 4 no data					M

Patients: 1 Records: 1 Filter: Off Sorted:

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
The screenshot shows the Audibase 5.2 Unregistered software interface. The window title is "Audibase 5.2 Unregist...". The menu bar includes "File", "Setup", "Edit", "Database", "Connect to", and "Help". The toolbar contains various icons, with the "Generate Graph" icon (a line graph) highlighted by a red box and a red arrow pointing to it from the text above. Below the toolbar, there are tabs for "Register", "Details", "Audiograms", and "Questionnaire". The main area displays a table with the following data:

Patient No	Surname	Forenames	Last Test	Time of Test	Previous Test	Date of Birth	Categorisation	Position/Rank	Dept/Unit	Company	Location	S
80171700	Luis	Casas	27/07/2018	12:09:58 pm	21/05/2018	16/03/1982	HSE 2006: 1 4 no data	Paciente			bogota	M

At the bottom of the window, there is a status bar showing "Patients: 1", "Records: 1", "Filter: Off", and "Sorted:". The Windows taskbar at the bottom of the screen shows the system tray with the date and time "12:05 p.m. 27/07/2018".

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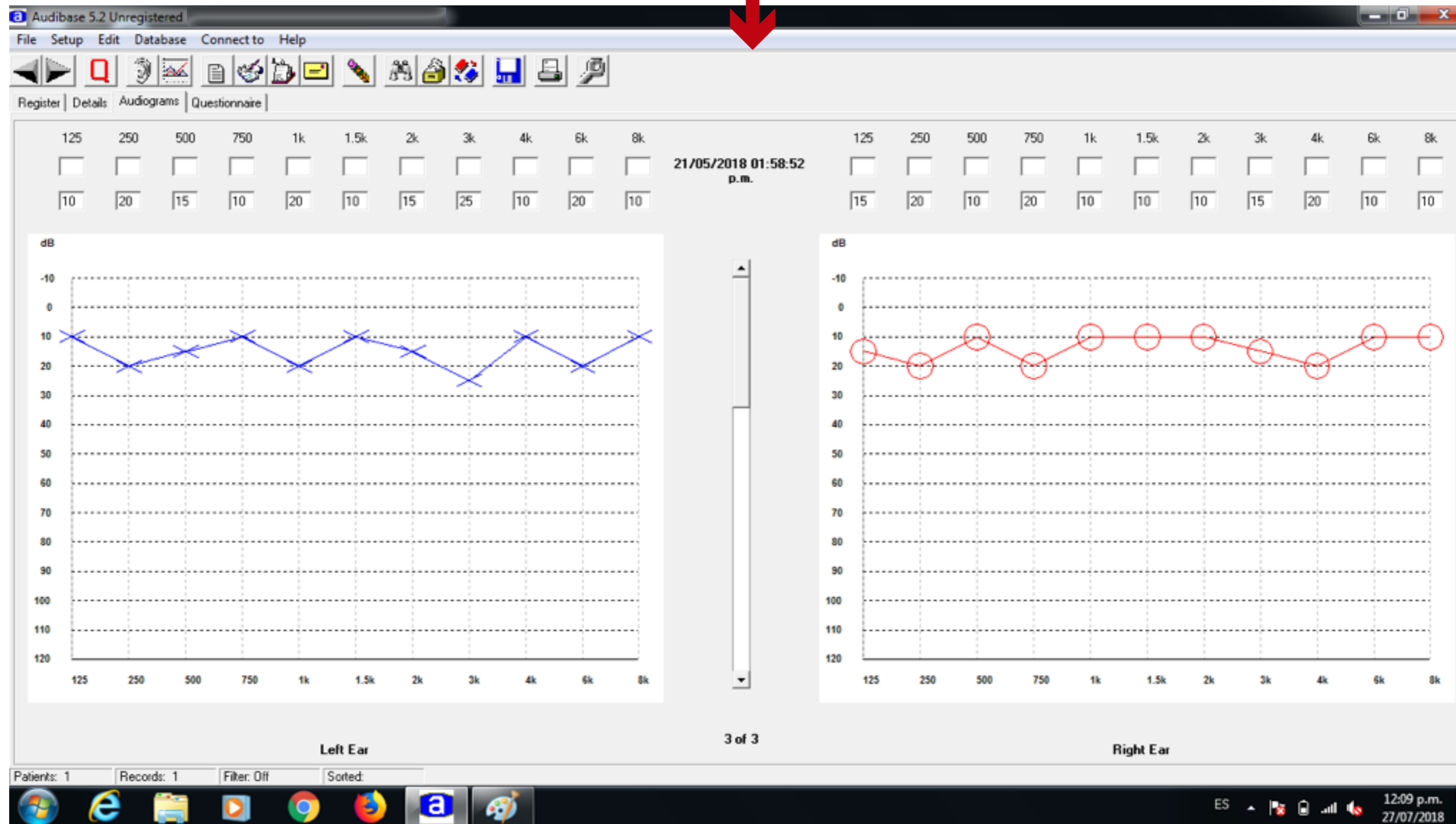Audibase 5.2 Unregistered

File Setup Edit Database Connect to Help

Register Details Audiograms Questionnaire

Patient No	Surname	Forenames	Last Test	Time of Test	Previous Test	Date of Birth	Categorisation	Position/Rank	Dept/Unit	Company	Location	S
80171720	Luis	Casas	27/07/2018	12:09:58 pm	21/05/2018	16/03/1982	HSE 2006: 1.4 no data	Paciente			bogota	M

Patients: 1 Records: 1 Filter: Off Sorted:

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